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Authorised Representative

DePuy Ireland UC
Loughbeg
Ringaskiddy
Co. Cork Ireland

Instruments and implants
approved by the AO Foundation.

 **DePuy Synthes**
PART OF THE JOHNSON & JOHNSON FAMILY OF COMPANIES

Instructions for Use

Please read these instructions for use, any enclosed information and relevant surgical techniques carefully before use. Ensure that you are familiar with the appropriate surgical technique.

The "TRAUMACEM™ V+ Syringe Kit, 4x1 mL, 2x2 mL" is a sterile packed, single use syringe kit used for the injection of DePuy Synthes PMMA-based cement into cancellous bone.

Since the "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL" is used in technically demanding procedures, the surgeon should be familiar with the use of the syringe kit as well as with the rest of the instrumentation and the procedure.

Product name and materials

"TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL". One kit contains the following parts:

Product name	Number of units	Materials
2 mL syringe, white plunger	2 units	PA12 (Polyamide) ABS (Acrylonitrile Butadiene Styrene) Silicone
1 mL syringe, blue plunger	4 units	PA12 (Polyamide) ABS (Acrylonitrile Butadiene Styrene) Silicone
Stop-cock	1 unit	PA12 (Polyamide) POM (Polyoxymethylen)

Intended use

The “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” is solely intended for the application of DePuy Synthes PMMA-based bone cement with the intent to augment cancellous bone. Refer to the corresponding directions regarding indications, contraindications, compatibility, use, precautions, warnings and side effects of the DePuy Synthes PMMA-based bone cement and access solution used in conjunction with the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL”. For information on compatibility with other devices or systems, consultation with a DePuy Synthes representative is recommended.

Side effects

There are no known side effects or interactions associated with the use of the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” together with DePuy Synthes PMMA-based bone cement.

Sterile device

STERILE	EO	Sterilized using ethylene oxide
----------------	-----------	---------------------------------

Store instruments in their original protective packaging, and do not remove them from the packaging until immediately before use. Prior to use, check the product expiration date and verify the integrity of the sterile packaging. If all packaging appear to be intact, remove product(s) from the package using aseptic technique.

Do not re-sterilize

Single-use device

Do not reuse

The “TRAUMACEM V+ Syringe Kit, 4×1 mL, 2×2 mL” is intended for single use only and must not be reused. Reuse or reprocessing (e.g. cleaning and/or resterilization) may compromise the structural integrity of the device and/or lead to device failure which may result in patient injury, illness or death.

Furthermore, reuse or reprocessing of single-use devices may create a risk of contamination, e.g. due to the transmission of infectious material from one patient or person to another. This could result in injury or death of the patient or user.

Device-specific warnings

- The purpose of the milliliter (mL) scale on the side of the “TRAUMACEM V+ Syringe Kit, 4×1 mL, 2×2 mL” syringes is to give physicians an indication of how much PMMA-based bone cement has been injected. This scale is for information purposes only.

Precautions

- The “TRAUMACEM V+ Syringe Kit, 4×1 mL, 2×2 mL” is used in technically demanding procedures. Therefore only physicians familiar with the proper technique and instrumentation used for the delivery and use of PMMA-based bone cement should use the “TRAUMACEM V+ Syringe Kit, 4×1 mL, 2×2 mL”.
- The “TRAUMACEM V+ Syringe Kit, 4×1 mL, 2×2 mL” should only be used to deliver PMMA-based bone cement in conjunction with the use of fluoroscopy imaging equipment capable of delivering high-quality images.

- Ensure a good fit between the "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL" syringe and the "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL" stop-cock/used access solution, but make sure to be on axis and avoid using excessive force when coupling them. They are both made of plastic and could otherwise break.
- After filling a "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL" syringe with cement, grip it by its wings. Gripping it anywhere else could transfer heat from your hand to the cement, shortening the working time of the PMMA-based bone cement.
- Always fill all "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL" syringes with PMMA-based bone cement directly after mixing.
- Always make sure the PMMA-based bone cement has reached the preferred viscosity prior to injection.
- To halt the flow of cement out of the "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL" syringe at any given time, simply stop pushing the plunger.
- Make sure to always inject cement in a slow, controlled fashion.
- If cement flow is hindered at any time during injection, halt, investigate and correct the reason for the flow hindrance.
- Do not use tools or excessive force to operate the "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL".
- The syringes in the "TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL" are solely intended to be used for PMMA-based cement augmentation and must not be used with radiographic contrast agent/contrast agent.
- For injection of radiographic contrast agent/contrast agent, only syringes with luer lock with an appropriate intended use should be used.
- Always consult the radiographic contrast agent/contrast agent instruction for use and the warnings therein prior to administra-

tion to ensure that the used radiographic contrast agent/contrast agent is approved for this application.

- CAUTION: Federal law restricts this device to sale in the US by or on the order of a physician.

Combination of medical devices

The “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” is tested and validated with the DePuySynthes PMMA-based bone cement and DePuy Synthes augmentation hardware only. DePuy Synthes has not tested compatibility with devices provided by other manufacturers and assumes no liability in such instances. For information on compatibility with other devices or systems, consultation with a DePuy Synthes representative is recommended. Refer to corresponding device’s or system’s directions for use.

Operating instructions

1. Prepare DePuy Synthes PMMA-based bone cement according to the corresponding directions of use and connect the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-cock using the side with no funnel. Ensure a good fit between “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-cock and cement mixing device but make sure to be on axis and avoid using excessive force when attaching them – they are both made of plastic and could otherwise break.
2. Next air has to be removed from the system. Hold the cement mixing device vertically and gently turn its handle clockwise. Note: To remove air, rotate the handle clockwise in a controlled manner – do not push it.
3. You will see the mixer piston advancing inside the translucent cartridge and a steady flow of cement will move into the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-

- cock. As soon as the cement starts to exit the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-cock at the funnel end, close it by turning the lever (“off”) 90°, toward the cement mixing device. This should stop the cement flow.
4. Attach a “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe to the stop-cock (funnel side). Ensure a good fit between the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-cock and the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe but make sure to be on axis and avoid using excessive force when attaching them – they are both made of plastic and could otherwise break.
 5. Open the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-cock by turning the lever 90° to the left, back to its original position. Turn the cement mixer handle clockwise in a slow and controlled fashion to fill the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe. PMMA-based bone cements can be highly viscous and need time to flow through the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-cock and into the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe. Make sure to fill the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe completely. Note: To transfer cement, rotate the handle clockwise in a controlled manner – do not push it.
 6. As soon as the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe is filled, stop the cement flow by again closing the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” stop-cock.
 7. Disconnect the filled “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe and attach the next one. Make sure to disconnect the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL”

syringe on axis, otherwise the connection could break. Repeat steps 4 through 7 until all “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringes are filled. Always fill all “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringes immediately after mixing.

8. Test the cement viscosity by ejecting cement from one of the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringes and evaluating it. Cement viscosity is an important factor in the safe delivery of cement to the intended operative site, as the risk for leakage decreases with increasing viscosity.
9. Once the viscosity is deemed appropriate by the responsible physician, the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe is attached to an access system and cement application can commence. Ensure a good fit between “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe and access solution but make sure to be on axis and avoid using excessive force when attaching them – “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringes are made of plastic and could otherwise break.
10. Exercise control when injecting PMMA-based bone cement. Always push the “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe piston on axis, otherwise the connection between “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringe and access solution could break.

The force necessary to inject cement increases with time, as the cement sets. The required force also increases with increased syringe size. It is therefore advised to start using the 2mL “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringes first and the 1mL “TRAUMACEM V+ Syringe Kit, 4x1 mL, 2x2 mL” syringes later.

Troubleshooting

If the force needed to inject cement becomes too high, this indicates that the maximum working time for the combination of DePuy Synthes PMMA-based cement and access solution used has been reached.

Device-related storage and handling information

Keep Dry

Keep away from sunlight

Additional device-specific information

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Synthes GmbH