## English

**G1 Dissection Tools:** Indications for Use: Anspach dissection tools are intended for cutting and shaping bone including spine and cranium. Sterile – Single Use Only. Do not re-sterilize or reuse. Reuse and/or re-sterilization of dissection tools could result in reduced performance, material degradation and/or patient contamination. For use with Anspach Systems only. For additional instructions refer to documents supplied with the relevant attachment and system being used.

**⚠ Warning:** Discard device and do not use if there is rust visible prior to use. Do not use with microMax, microMax Plus or BlackMax Handpieces. Surgeon is responsible for learning proper techniques in use of this equipment, as improper use may cause serious injury. All operating room personnel must wear eye protection. Delicate structures in proximity to dissection must be thoroughly protected to prevent injury. Confirm attachment is proper size for dissection tool. Dissection tools must be adequately retained within handpiece to prevent distal migration, which may cause injury. Do not force dissection tools during use. Forceful side loading may cause them to fracture, which may cause injury. Do not use if bent. Do not attempt to straighten if bent as that may cause failure. Irrigation is mandatory for proper operation. Properly dispose of in accordance with medical center, local and national government requirements and laws. Confirm twist drill with depth stop is properly sized for depth of cut. Latex Information: Not made with natural rubber latex.

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© DePuy Synthes 2019. All rights reserved.
**The Anspach Effort, Inc.**
**Synthes GmbH**