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### English microSaws and Small Attachments

#### List of Devices

The following instructions apply to the devices listed below.

BM-DRIVER	O-SAW	S-SAW	SA-JLATCH
DRIVER	R-SAW	SA-JACOBS	

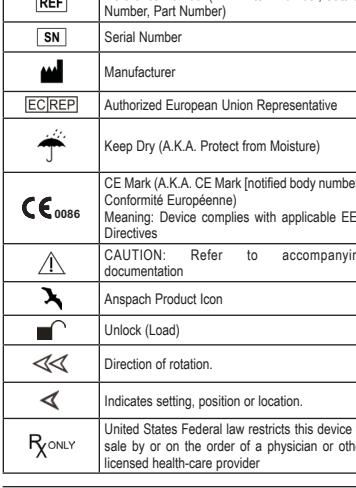
#### Indications For Use

Intended for cutting and shaping bone including spine and cranium.

#### Cleaning and Sterilization

- Prior to first use equipment must be processed following the Cleaning and Sterilization instructions accompanying this device.
- At the point of use the device must be cleaned as soon as possible after use to prevent drying of blood, tissue, other biological debris and contaminants on the device.

#### Symbols

Note: The following instructions are written such that the handpiece and attachments are held with distal end pointed away from user.

#### XMax, microMax, and eMax Keyless Driver (DRIVER)

- ##### Assembly
- Ensure knurled knob on handpiece is in "Run" or "Secure" position.
  - Insert drive shaft of keyless driver into handpiece and rotate clockwise until locked. Gently twist back and forth to ensure it is locked into position.

- ##### Disassembly
- Pull knurled knob down and twist counterclockwise until it seats into position.
  - Rotate DRIVER counter-clockwise and remove.

#### Black Max Keyless Driver (BM-DRIVER)

- ##### Assembly
- Ensure locking lever is up.
  - Align notch on keyless driver with "Release" arrow on handpiece while inserting drive shaft of Keyless Driver into handpiece. Rotate keyless driver clockwise until notch is aligned with "Secure" arrow on handpiece. Push locking lever down to locked position.

- ##### Disassembly
- Raise locking lever.
  - Rotate BM-DRIVER counter-clockwise and remove from handpiece.

#### Oscillating Saw (SAW), Sagittal (S-SAW), and Reciprocating (R-SAW) Saw Attachments

- ##### Assembly
- Insert attachment drive shaft into distal end of keyless driver. Rotate until three slots on attachment engage three slots in keyless driver.
  - Continue to rotate counterclockwise while pushing attachment into keyless driver until attachment locks into position and release ring snaps into position. When fully engaged, contours of the keyless driver body, release ring, and saw attachment will be in alignment.

- ##### Disassembly
- With one hand supporting attachment, rotate release ring on keyless driver with other hand counterclockwise until attachment is released.

#### Oscillating Saw Blade Assembly

- Press down on release button to open saw blade mounting plates.
- Insert saw blade into opening thru slot on mounting plate and align saw blade with locking pins in mounting plate.
- Release button and ensure mounting plate is firmly seated on saw blade without gaps.

#### Sagittal Saw Blade Assembly

- Push down on release button to open saw blade mounting plate.
- Insert saw blade into opening thru slot on mounting plate and align saw blade with locking pins in mounting plate.
- Release button and ensure mounting plate is firmly seated on saw blade without gaps.

#### Reciprocating Saw Blade Installation

- Rotate chuck counterclockwise until chuck has opened sufficiently to accept a saw blade.
- If blade has a shaft, insert shaft of blade in chuck jaws until it is fully inserted and tighten chuck knob until snug.
- If blade has a flat shaft, insert blade into slots in chuck until it is fully inserted and tighten chuck knob clockwise until snug.

#### Saw Blade Removal

1. Saw blade removal is reverse of installation.

#### J-Latch (SA-JLATCH) and Keyless Jacobs Chuck (SA-JACOBS) Attachments

- Note: Keyless driver stands for either DRIVER or BM-DRIVER.

##### Assembly

- Insert attachment drive shaft into distal end of keyless driver. Rotate until three slots on attachment engage three slots in keyless driver.
- Continue to rotate counterclockwise while pushing attachment into keyless driver until attachment locks into position and release ring snaps into position.
- When fully engaged, contours of the keyless driver body, release ring, and small attachment will be in alignment.

- ##### Disassembly
- With one hand supporting attachment, rotate release ring on keyless driver with other hand counterclockwise until attachment is released.

#### J-Latch (SA-JLATCH) Dissection Tools

- Note: Synthes Anspach does not supply dissection tools for SA-JLATCH small attachments.

##### Assembly

- Push down on center of release button located on SA-JLATCH attachment and hold it down to insert dissection tool into distal end of attachment.
- Rotate dissection tool slowly until fully seated.
- Release distal sleeve and check security of dissection tool by pulling distally on dissection tool.
- Dissection tool and attachment are secure.

- ##### Disassembly
- Push back distal sleeve in the direction of arrow V on SA-JLATCH attachment and hold it down while pulling dissection tool out distally for removal.
  - Release the distal sleeve.

#### Keyless Jacobs Chuck (SA-JACOBS) Dissection Tools

- Note: Synthes Anspach does not supply dissection tools for SA-JACOBS small attachments.

##### Assembly

- Press down on center of release button located on SA-JACOBS attachment while rotating knurled knob on attachment clockwise until fully opened.
- Inset dissection tool into distal end of attachment until it touches bottom.

3. Rotate knurled knob counterclockwise while centering cutter on attachment until tight; release button.

4. Check security of dissection tool by pulling distally on dissection tool.

5. Dissection tool and attachment are secure.

Do not press down on release button while attachment is in use.

6. Press down on center of release button located on SA-JACOBS attachment while rotating knurled knob on attachment clockwise until fully opened.

7. Pull dissection tool out distally for removal.

8. Release button.

- ⚠ Warning: Do not operate without irrigation which is mandatory for proper operation.

#### CAUTIONS

- Surgeon is responsible for learning proper techniques in use of equipment; improper use may cause serious injury to user or patient or damage to system. Instrument operator and all operating room personnel must wear eye protection. • Visually inspect for damage before using; do not use if damage is evident. • Dispose of items contaminated with body fluids or other biohazardous waste. • At end of life cycle recycle or dispose of device in accordance with local and national regulations. • Do not use if the product sterilization barrier or its packaging is compromised. • Do not use, or discontinue use of powered equipment exhibiting excessive temperatures that can cause patient injury (necrosis) and/or user discomfort. • Use of damaged or incomplete equipment can result in excessive temperatures. • Use of damaged equipment can result in excessive temperatures. • Use caution to avoid cutting or tearing gloves while handling dissection tools.

- Dissection tools must be adequately retained within attachment to prevent distal migration, which may cause injury. • Confirm attachment is proper size for dissection tool and that it is secure. • Gently pull on dissection tool shaft to ensure it is fully seated and properly installed.

- Only cut visible areas unless an image intensifier is utilized.

- Delicate structures in proximity to dissection must be thoroughly protected to prevent injury. • Maintain firm control of instrument at all times. • Do not bend or use as a lever. • Use a gentle tapping motion or side-to-side motion and let instrument do cutting. • Do not use excessive force. • Forceful side loading of dissection tool may cause fracture of dissection tool or cause injury. • Dissection tool should be dispensed for single use only. • Do not re-sterilize and/or re-use dissection tools. • Use standard protocol for disposal of sharp instruments. • Continuous extreme cutting at or near stalling conditions will quickly overheat handlepiece. • Do not allow liquid into console. • Do not use in oxygen rich environment.

- To avoid the risk of electric shock, this equipment must only be connected to a supply mainly with protective earth. • Do not operate in an explosive flammable environment. • Do not modify ground or power cord. • Portable and mobile RF communications equipment can affect Medical Electrical Equipment. • Use of accessories or cables other than those provided by Synthes Anspach, and specified for the electric system in use, may result in increased emissions or decreased immunity.

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