English

Otologic Curved Micro (OCM) Attachment and Otologic Curved Micro (OCM) Burr Support Sleeves

List of Devices
The following instructions apply to the devices listed below.

OCM	OCMS-1SD	OCMT-0SD	OCMT-1SD
OCMS-0SD	OCMS-1SB	OCMT-0SD	OCMT-2SD
OCMS-1SD	OCMS-1SD	OCMT-0SD	OCMT-2SD
OCMS-0SD	OCMS-2SD	OCMT-1SB	
OCMS-0SD	OCMS-2SD	OCMT-1SB	
OCMS-1SD	OCMS-1SD	OCMT-1SD	
OCMS-1SD	OCMS-1SD	OCMT-1SD	

Device Description
The OCM Curved Micro (OCM) Attachment connects to XMax® Pneumatic and eMax 2 and eMax 2 Plus Electric Systems and accepts OCM Burr Support Sleeves which are available in various cutting burr types.

The OCM Burr Support Sleeve (OCM) consists of the cutter and an outer sleeve that protects and supports the rotating shaft of the cutting burr.

Warning: Device must be cleaned prior to sterilization using a validated cleaning method from instructions provided with device in accordance with ISO 17664 and ANSI/AAMI ST1.

1. Packaging for Sterilization:

With the sterilization wrap the sterilization container is to be used.

• Ensure sterilization equipment is in proper working order as specified by manufacturer.

• Manufacturer's instructions are properly employed by trained and qualified personnel.

• Assume actual cycle employed has been properly validated for the device(s)/tool configuration being processed and appropriate sterilization indicator devices are included for each product and cycle.

Warning: Device must be cleaned prior to sterilization using a validated cleaning method from instructions provided with device in accordance with ISO 17664 and ANSI/AAMI ST1.

2. Sterilization:

• Sterilize according to the following guidelines.

Warning: Do not use any other sterilization method or cycle than those listed below which have been validated in accordance with ISO 17665.

3. Indications for Use

When used with the ANSPACH® Systems, the OCM Attachment and OCM Burr Support Sleeves are intended for cutting and shaping bone primarily in otology procedures such as cochleostomies.

Symbols

REF	Reference Number (A.K.A. Item Number, Catalog Number, Part Number)
SN	Serie Number
Manufacturer	
[EC REP]	Authorized European Union Representative
Keep Dry (A.K.A. Protect from Moisture)	
CE 0086	CE Mark (A.K.A. CE Marked (notified body number), Conformité Européenne) Measuring: Device complies with applicable EEC Directives
Ansprech Person	Ansprech Person
Consult Operating Instructions	Consult Operating Instructions
CAUTION: Refer to accompanying documentation	CAUTION: Refer to accompanying documentation
Lock (Run)	Lock (Run)
Indicates direction of rotation for lock position.	Indicates direction of rotation for lock position.
R ONLY	United States Federal law restricts this device to sale by or on the order of a physician or other licensed health-care provider

Warning: Prior to use the OCM Attachment must be processed as per the included Cleaning and Sterilization instructions.

• Prior to use the OCM Attachment must be processed as per the included Cleaning and Sterilization instructions.

• It is not intended for gross bone removal. Heavy leading and/or lack of irrigation while performing bone dissection may result in bone fracture.

• Prior to use inspect the instrument(s). Do not use the instrument(s) if damaged.

• To avoid overheating this device, employ a duty cycle of 10 seconds ON and 10 seconds OFF for unlimited number of cycles.

Note: When following these instructions hold handpiece with distal end pointing away from user.

OCM Installation for XMax and eMax Handpiece:

1. Ensure handpiece is in "RUN" or "SECURE" position.

2. Insert OCM Attachment's drive shaft into distal end of handpiece, push and rotate to right until it stops. Listen for a click as handpiece moves to secure the OCM Attachment.

3. Once handpiece is locked to secure OCM Attachment is locked into position.

OCM Burr Support Sleeve Disassembly:

1. Remove OCM Burr Support Sleeve by grasping OCM Burr in grooves. (Illustration 1)

2. Treat the OCM Burr Support Sleeve as bioburden with bioplates of accordingly

OCM Disassembly for XMax and eMax Handpiece:

1. Move handpiece knurl to "SAFE" position.

2. Rotate OCM Attachment to left approximately one quarter turn, and remove from distal end of handpiece housing.

Manual Cleaning & Lubrication

Warnings:

• Device must be cleaned as soon as possible after use to prevent drying of blood, tissue, other biological debris and contaminants on the device.

• Not to use on skin or mucous membranes.

• You must follow standard hospital procedures.

• You must follow manufacturer's recommendations for any detergents, disinfectants and cleaning agent used.

• Prior to use, clean the OCM Burr Support Sleeves are available and SINGLE USE ONLY. Reuse and reprocessing of dissection tools could result in reduced performance, material degradation and/or patient contamination.

Cautions:

• Do not use an alkaline detergent with a pH greater than 11. Alkaline detergents with higher pH attack grease and sealants which can increase wear and lead to malfunction.

• Fully dechlorinated (or purified) water must be used during final rinse to prevent corrosion and spots.

• Avoid exposure to hypochlorite solutions (i.e. bleach), as these will promote corrosion.

Note: Use of alkaline detergents can cause device color to fade but this does not impair function.

Operating Room (OR) Personal Instructions

1. After completion of procedure, remove external debris by wiping attachments and handpiece.

2. Dispose of open use dissection tools.

3. Reprocessing Personnel Instructions

1. Preparation

• Always assemble all necessary supplies including Attachment Cleaning Brush (ACB), sponge, lint-free cloth and soft cloth brush.

• Remove OCM Attachment and OCM Burr from handpiece.

2. Rinse

• Rinse OCM Attachment under running cold deionized (DI) or purified water (PURW) for a minimum of 2 minutes.

• Manipulate any given parts such as release sleeves under running cold deionized (DI) or purified water (PURW) to loosen any hard tissue or gross soil.

• Use a sponge, soft lint-free cloth or soft brush cloth to assist in removing gross soil.

Caution: Do not use high pressure water or air. Do not rinse OCM Attachment with saline solution.

3. Clean

• Prepare cleaning solution using alkaline detergent (non-chlorinated) or neutral pH enzymatic detergent per manufacturer's directions for correct temperature, water quality (i.e. pH, hardness) and product concentration/ dilution.

• Fully immerse OCM Attachment in cleaning solution as per manufacturer's instructions and manually agitate for 2 minutes.

• Use a sponge, soft lint-free cloth or soft brush cloth to remove all visible soil.

• Gently use ACB to clean cannulations of OCM Attachment while immersed in cleaning solution.

Warning: OCM is single use and should only be used to clean cannulations.

4. Rinse

• Prepare cleaning solution using alkaline detergent (DI) or purified water (PURW) for a minimum of 2 minutes.

• Use a sponge or piece of cloth with deionized (DI) or purified water (PURW) to flush lumens or channels.

Caution: Do not use high pressure water or air. Do not rinse OCM Attachment with saline solution.

5. Inspect

• Visually inspect OCM Attachment for visible soil. Repeat Steps 2 thru 5 if visible soil remains.

6. Lubricate

• Lubricate this step for the OCM attachment only. Do not lubricate handpieces.

• Option 1: 0.05:001:078 Lubricant for Ansprech Systems, 10 ml

Warning: Do not use Gravity Air Dispenser System/autoclave sterilization Process.

• This step is for the OCM attachment only. Do not use the OCM Attachment and one (1) to the proximal end of the OCM Attachment.

• Applying excess amounts of lubricant will cause the drip from the attachment. Clean off any excess lubricant.

• Remove OCM Attachment and allow it to drain completely until no visible droplets are coming from it.

Caution: Do not reuse instrument milk (lubricant). Do not apply mineral oil or other lubricants, which may cause OCM Attachment to overheat.

Note: The clinical processing instructions provided have been validated by DePuy Synthes Power Tools for preparing a non-sterile medical device; this instruction is provided in accordance with ISO 17664 and ANSI/AAMI ST1.

Option 2: Non-Silicone Based Medical Lubricant

• Prepare a lubricating solution of instrument milk (non-silicone based medical lubricant) per the manufacturer's directions.

• Fully immerse OCM Attachment in lubricating solution as per manufacturer instructions, at room temperature, and agitate for 15 seconds.

• Remove OCM Attachment and allow it to drain completely until no visible droplets are coming from it.

Caution: Do not reuse instrument milk (lubricant). Do not apply mineral oil or other lubricants, which may cause OCM Attachment to overheat.

Note: The clinical processing instructions provided have been validated by DePuy Synthes Power Tools for preparing a non-sterile medical device; this instruction is provided in accordance with ISO 17664 and ANSI/AAMI ST1.

DePuy Synthes Power Tools, Division of DJO 2016.

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