## Irrigation Devices

<b>System Description</b> Irrigation tubing, attachment clips and hose clips.		
IRR-TUBE	IRR-CLIP-20	IRR-CLIP-50
IRR-TUBE-HF	IRR-CLIP-30	IRR-CLIP-60
IRR-CLIP-10	IRR-CLIP-40	HOSECLIPS-G1

### Indications for Use

- Anspach® Surgical Irrigation Systems are indicated for use with Anspach Surgical Motor Systems for providing controlled, cooling irrigation during cutting, shaping and removal of bone, including bones of the skull and spine.
- The attachment clips are intended to position and hold the irrigation tubing to the attachment and handpiece/motor during surgical procedures where irrigation is being used.
- The irrigation hose clips are intended to help hold the irrigation tubing and the handpiece hose together during surgical procedures where irrigation is being used.

### Assembly Instructions

Refer to eMax 2 and eMax 2 Plus System directions for irrigation set up and operation.

**Warning:** For proper operation only use at ambient temperatures between 65°F to 85°F (18°C to 29°C).

### Cleaning and Sterilization

- Sterile Packaged
- Disposable.
- Single use only.

**Warning:** Reuse and/or re-sterilization of these devices could result in reduced performance, material degradation and/or patient contamination.

### Disposal

Properly dispose of in accordance with medical center, local and national government requirements and laws.

### Latex Information

Not made with natural rubber latex.

### Warnings

Surgeon is responsible for learning proper techniques in use of equipment; improper use may cause serious injury to user or patient or damage to system.
• Instrument operation and all operating room personnel must wear eye protection.
• Inspect system for damage before using; do not use if damage is evident.
• Disposable tubing, attachment clip, and hose clip are for SINGLE USE ONLY.

### Cautions

Rx Only: United States Federal law restricts this device to sale by or on order of a physician or other licensed health-care provider.
• Do not use accessories other than those provided by DePuy Synthes Power Tools and specified for use with Anspach Systems.
• To ensure equipment operates as designed, read and follow manufacturer’s instructions.

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