Irrigation Devices

English

System Description

Irrigation tubing, attachment clips and hose clips.

IRR-TUBE	IRR-CLIP-10	IRR-CLIP-30	IRR-CLIP-50
IRR-TUBE-HF	IRR-CLIP-20	IRR-CLIP-40	IRR-CLIP-60

Indications for Use

- Anspach® Surgical Irrigation Systems are indicated for use with Anspach Surgical Motor Systems for providing controlled, cooling irrigation during cutting, shaping and removal of bone, including bones of the skull and spine.
- The attachment clips are intended to position and hold the irrigation tubing to the attachment and handpiece/motor during surgical procedures where irrigation is being used.
- The irrigation hose clips are intended to help hold the irrigation tubing and the handpiece hose together during surgical procedures where irrigation is being used.

Assembly Instructions

Refer to eMax 2 and eMax 2 Plus System directions for irrigation set up and operation.

Warning: For proper operation only use at ambient temperatures between 65°F to 85°F (18°C to 29°C).

Cleaning and Sterilization

- Sterile Packaged
- Disposable.
- ⊗ Single use only.

Warning: Reuse and/or re-sterilization of these devices could result in reduced performance, material degradation and/or patient contamination.

Disposal

Properly dispose of in accordance with medical center, local and national government requirements and laws.

Latex Information

Not made with natural rubber latex.

Warnings:

Surgeon is responsible for learning proper techniques in use of equipment; improper use may cause serious injury to user or patient or damage to system.
• Instrument operator and all operating room personnel must wear eye protection.
• Inspect system for damage before using; do not use if damage is evident.
• Disposable tubing, attachment clip, and hose clip are for SINGLE USE ONLY.

Cautions

Rx Only: United States Federal law restricts this device to sale by or on order of a physician or other licensed health-care provider.
• Do not use accessories other than those provided by DePuy Synthes Power Tools and specified for use with Anspach Systems.
• To insure equipment operates as designed, read and follow manufacturer’s instructions.

Manufacturer

The Anspach Effort[®], Inc.

4500 Riverside Drive
Palm Beach Gardens, FL 33410 USA
Tel: (800) 327 6887 / +1 561 627 1080
Fax: (800) 327 6661 / +1 561 625 9110
Web: www.depuyssynthes.com
Email: infoPowerTools@DPYUS.jnj.com

EC**RE****P**

Synthes GmbH

Eimattstrasse 3
4436 Oberdorf
Switzerland
Tel. +41 61 965 61 11

For patent information go to http://www.depuyssynthes.com/patentmarking.
© DePuy Synthes Power Tools, Division of DOI 2016.

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