# **G1 High Speed Attachments Assembly and Disassembly**

## Contents

General Information.....	2
Straight Attachments .....	6
Craniotome Attachments .....	7
QD Angle Attachments .....	8
Minimal Access Attachment and Bearing Sleeves.....	10
Perforator Driver Attachment.....	12

## General Information

### Device Description

Attachments designed for use with the Anspach™ G1 High Speed System and G1 Dissection Tools.

### Indications for Use

Cutting and shaping bone including spine and cranium.

### System Assembly

Refer to the Anspach eG1 High Speed System instructions accompanying the specific device.

### Cleaning and Sterilization

- Prior to first use the equipment must be processed as per the included Anspach G1 High Speed System Cleaning and Sterilization instructions.
- At the point of use the device must be cleaned as soon as possible after use to prevent drying of blood, tissue, other biological debris and contaminants on the device.
- Refer to the Anspach G1 High Speed System Cleaning and Sterilization instructions accompanying this device.

### Attachment and Dissection Tool Selection

To assist in identifying the correct attachment and dissection tool combination, attachments have color bars at proximal end that correspond with color bars on dissection tool package labels as shown in Table 1 below.

**Table 1**

Attachment	Color Bars	Category
SHORT-G1	YELLOW / YELLOW	Straight Attachment
QD8-G1	BLACK / YELLOW	QD Angle Attachment
QD8-S-G1	BLACK / YELLOW	QD Angle Attachment
MEDIUM-G1	ORANGE / ORANGE	Straight Attachment
QD11-G1	BLACK / ORANGE	QD Angle Attachment
QD11-S-G1	BLACK / ORANGE	QD Angle Attachment
LONG-G1	RED / RED	Straight Attachment
QD14-G1	BLACK / RED	QD Angle Attachment
QD14-S-G1	BLACK / RED	QD Angle Attachment
MIA16-G1	BLACK / BLACK	Straight Attachment
SHORT-HD-G1	LIGHT BLUE / GREEN	Straight Attachment
MEDIUM-HD-G1	LIGHT BLUE / YELLOW	Straight Attachment
LONG-HD-G1	LIGHT BLUE / BLUE	Straight Attachment
XL-HD-G1	LIGHT BLUE / BLACK	Straight Attachment
CRANI-A-G1	GREEN / GREEN	Craniotome Attachment
CRANI-P-G1	TURQUOISE / TURQUOISE	Craniotome Attachment

Attachment	Color Bars	Category
CRANI-L-G1	GOLD / GOLD	Craniotome Attachment
MA-D20-G1	NA	Minimal Access Attachment
CSR60-G1	NA	Perforator Driver Attachment

## Symbols

	Anspach Product Icon		Reference Number (A.K.A. Item Number, Catalog Number, Part Number)
	Indicates the attachment setting, refer to MA-D20-G1 section in this document for further details.		Lot (A.K.A. Lot Number, Batch Number, Batch Code)
	Indicates position, alignment, or location. (Color: Black or Red)		Serial Number
	Indicates position or location		Temperature Limits
	Direction of rotation		Use-by Date (A.K.A. Expiration Date, Expiry)
	Run; System ready to operate.		Manufacturer
	Load; Insert attachment and/or burr. System will not operate.		Authorized European Union Representative
	Consult Operating Instructions		Sterilized Using Irradiation
	CAUTION: Refer to accompanying documentation		Single Use Only (A.K.A. Do Not Reuse)
	Indicates position, alignment, or location. (Color: Black or Red)		Keep Dry (A.K.A. Protect from Moisture)
	DO NOT DISPOSE OF IN HOUSEHOLD WASTE		Sterile unless damaged or open

R <sub>X</sub> ONLY	United States Federal law restricts this device to sale by or on the order of a physician or other licensed health-care provider	CE 2797	CE Mark (A.K.A. CE Mark [notified body number], Conformité Européenne) Meaning: Device complies with applicable EEC Directives
	Date of Manufacture	(01)	Global Trade Item Number (GTIN)
(10)	Batch or Lot Number	(17)	Expiration Date (YYMMDD)
(21)	Serial Number		Quantity
GTIN	Global Trade Item Number		

## Warnings

- **⚠ Always use continuous irrigation to prevent heat build-up. Irrigation is necessary for proper performance.**
- Prior to first use the equipment must be processed as per the included Anspach G1 High Speed System Cleaning and Sterilization instructions.
- Do not use a dissection tool without an attachment, injury may occur.
- Only Use G1 Dissection Tools. Using other than indicated dissection tools may lead to injury.
- Use the indicated dissection tool for the attachment, injury may occur.
- Surgeon is responsible for learning proper techniques in use of equipment; improper use may cause serious injury to user or patient or damage to system.
- Instrument operator and all operating room personnel must wear eye protection.
- Visually inspect for damage before using; do not use if damage is evident.
- Do not use if the product sterilization barrier or its packaging is compromised.
- Do not use or discontinue use of powered equipment exhibiting excessive temperatures that can cause patient injury (necrosis) and/or user discomfort.
- Use of damaged or improperly maintained power equipment and/or misused powered equipment can result in excessive temperatures.
- Use caution to avoid cutting or tearing gloves while handling dissection tools.
- Dissection tools must be adequately retained within attachment to prevent distal migration, which may cause injury. Confirm attachment is proper size for dissection tool and that it is secure.
- Gently pull on dissection tool shaft to ensure it is fully seated and properly installed.
- Only cut visible areas unless an image intensifier is utilized.
- Delicate structures in proximity to dissection must be thoroughly protected to prevent injury.

- Maintain firm control of instrument at all times.
- Do not bend or use as a lever.
- Use a gentle tapping motion or side-to-side motion and let instrument do cutting.
- Do not use excessive force.
- Forceful side loading of dissection tool may cause fracture of dissection tool, which may cause injury.
- Use standard protocol for disposal of sharp instruments.
- Dissection tools are disposable and intended for single patient use only. Do not re-sterilize and/or re-use dissection tools.
- Continuous extreme cutting at or near stalling conditions will quickly overheat handpiece.
- Do not operate in an explosive flammable environment.
- Do not modify ground or power cord.
- Do not allow liquid into console.
- Use of accessories or cables other than those provided by DePuy Synthes Power Tools and specified for use with eG1 System may result in increased emissions or decreased immunity.
- The eG1 should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, the eG1 should be observed to verify normal operation in the configuration in which it will be used
- Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in this accompanying documentation.
- Portable and Mobile RF communications equipment can affect Medical Electrical Equipment.
- Power source should comply with applicable IEC, CEC, and NEC requirements. Grounding reliability can only be achieved when this equipment is connected to a receptacle marked "HOSPITAL GRADE."
- Do not use in oxygen rich environment.
- No modification of this equipment is allowed.
- Do not modify. Modifications could result in loss of electrical safety.
- Dispose of items contaminated with body fluids with other biohazardous waste.
- At end of life recycle or dispose of device in accordance with local and national regulations.
- To avoid the risk of electric shock, this equipment must only be connected to a supply main with protective earth. The use of Accessories, transducers, and cables other than those specified, with the exception of transducers and cables sold by the Manufacturer of this device as replacement parts for internal components, may result in increased Emissions or decreased Immunity of the eG1 System.

## Cautions

- United States Federal law restricts this device to sale by or on order of a physician or other licensed healthcare provider.
- Do not use handpiece without an attachment and dissection tool properly locked in place. Damage to the burr lock mechanism could occur.

## English

- Do not use accessories other than those provided by DePuy Synthes Power Tools and specified for use with Anspach Systems.
- To ensure equipment operates as designed, read and follow manufacturer's instructions.
- Do not operate handpiece without an attachment and the corresponding dissection tool.
- Only Anspach G1 Dissection Tools should be used with G1 Systems.
- Use care to protect hose when handling, cleaning, and during system use.
- Damage to hose can cause leaking, rupture, or other related failures.
- Do not step on, set equipment on, pinch, kink, clamp, or otherwise occlude handpiece hose during use.

### Latex Information

Not made with natural rubber latex.

### Warranty & Return Policies

Warranty and return policy is available upon request.

**Warning:** All Instrument System components returned for servicing or repair should be properly cleaned and sterilized as applicable prior to shipping.

### Warning: Transmissible Spongiform Encephalopathies (TSE)

DePuy Synthes Power Tools will not authorize or accept the return of products that directly contact patients or are contaminated with a patient's body fluids who is suspected or confirmed with a Transmissible Spongiform Encephalopathies/ Creutzfeldt-Jakob Disease (TSE/CJD) diagnosis. DePuy Synthes Power Tools recommends that all Anspach products used on a patient confirmed with a TSE/CJD diagnosis be incinerated. Anspach dissection tools used on a patient suspected of TSE/CJD diagnosis must be incinerated.

Contact your Sales Representative for replacement of product incinerated under this policy or for temporary equipment while original equipment is quarantined. Contact the DePuy Synthes Power Tools Customer Service Department regarding TSE/CJD contamination for additional information.

### Straight Attachments

**Note:** When following these instructions hold handpiece and attachments with distal end pointing away from user.

### Assembly

1. Confirm indicator bar on distal end of handpiece is aligned with unlock symbol (Figure 1) before attempting to install attachment. If not aligned properly, hold handpiece housing firmly with one hand and with other hand rotate distal end in direction of arrow to align indicator bar with unlock symbol.
2. Slide attachment over distal end of handpiece aligning arrow on attachment with indicator bar and unlock symbol (Figure 2). Pull back firmly until fully seated against handpiece.
3. Rotate attachment in direction of arrow on handpiece to align arrow on attachment with lock symbol (Figure 3).

4. Insert dissection tool into distal end of attachment and rotate dissection tool slowly until fully seated.
5. Gently pull on dissection tool to ensure proper engagement.

**Note:** Handpiece is now fully functional.

## Disassembly

1. Rotate attachment to align arrow with unlock symbol (Figure 4).
2. Remove dissection tool from attachment.
3. Remove attachment from handpiece.

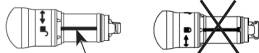
Figure 1 Indicator Bar

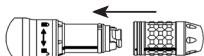
Figure 2

Figure 3

Figure 4

## Craniotome Attachments

**Note:** When following these instructions hold handpiece and attachments with distal end pointing away from user.






### Assembly

1. Confirm indicator bar on distal end of handpiece is aligned with unlock symbol (Figure 1) before attempting to install attachment. If not aligned properly, hold handpiece housing firmly with one hand and with other hand rotate distal end in direction of arrow to align indicator bar with unlock symbol.
2. Insert dissection tool into distal end of handpiece and rotate dissection tool slowly until fully seated. (Figure 2).
3. Slide attachment over distal end of dissection tool and handpiece aligning arrow on attachment with indicator bar and unlock symbol (Figure 3). Pull back firmly until fully seated against handpiece.
4. Rotate attachment in direction of arrow on handpiece to align arrow on attachment with lock symbol (Figure 4).
5. Visually inspect to confirm dissection tool is not touching attachment guard.

**Note:** Handpiece is now fully functional.

## Disassembly

1. Hold handpiece firmly in one hand and with other hand push and rotate release sleeve of attachment until arrow is aligned with unlock symbol (Figure 5).
2. Remove attachment from handpiece.  
**Warning:** Use caution to avoid cutting hands or fingers on dissection tool when performing this step.
3. Remove dissection tool from attachment.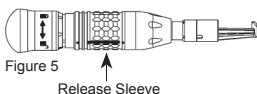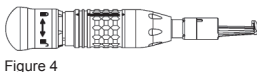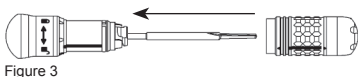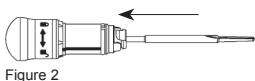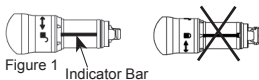

## QD Angle Attachments

### Warning

Observe the following duty cycle to avoid overheating of this device.

- Employ a duty cycle of 20 seconds ON and 40 seconds OFF for an unlimited number of cycles at ambient temperatures up to 29°C (85° F).

**Note:** When following these instructions hold handpiece and attachments with distal end pointing away from user.

### Assembly







1. Confirm indicator bar on distal end of handpiece is aligned with unlock symbol (Figure 1) before attempting to install attachment. If not aligned properly, hold handpiece housing firmly with one hand and with other hand rotate distal end in direction of arrow to align indicator bar with unlock symbol.

- Slide attachment over distal end of handpiece aligning arrow on attachment with indicator bar and unlock symbol (Figure 2). Pull back firmly until fully seated against handpiece.
- Rotate attachment in direction of arrow on handpiece to align arrow on attachment with lock symbol (Figure 3).
- Rotate attachment lock sleeve to align arrow with lock symbol (Figure 4).
- Insert dissection tool into distal end of attachment and rotate dissection tool slowly until fully seated.
- Gently pull on dissection tool to ensure proper engagement.

**Note:** Handpiece is now fully functional.

### Disassembly

- Rotate attachment lock sleeve to align arrow with unlock symbol (Figure 5).
- Remove dissection tool from attachment.
- Hold handpiece firmly in one hand and with other hand push and rotate release sleeve of attachment until arrow is aligned with unlock symbol (Figure 6).
- Remove attachment from handpiece.

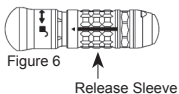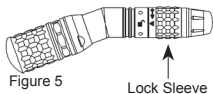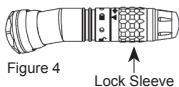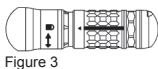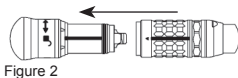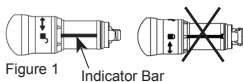## Minimal Access Attachment and Bearing Sleeves

### Warning

Observe the following duty cycle to avoid overheating of this device.

- Employ a duty cycle of 20 seconds ON and 40 seconds OFF for an unlimited number of cycles at ambient temperatures up to 29°C (85° F).

**Note:** When following these instructions hold handpiece and attachments with distal end pointing away from user. Minimal Access Attachments are compatible with multiple bearing sleeve configurations. Instructions are the same for all bearing sleeve configurations.

The four words labeled on attachment near lock sleeve (Figure 1) mean the following:

- CUTTER**— Insert or remove dissection tool.
- RUN**— Attachment and dissection tool are now functional.
- ADJUST**— Bearing sleeve can be moved to desired exposure.
- TUBE**— Insert or remove bearing sleeve.

### Assembly

1. Confirm indicator bar on distal end of handpiece is aligned with unlock symbol (Figure 2) before attempting to install attachment. If not aligned properly, hold handpiece housing firmly with one hand and with other hand rotate distal end in direction of arrow to align indicator bar with unlock symbol.
2. Slide attachment over distal end of handpiece aligning arrow on attachment with indicator bar and unlock symbol (Figure 3). Pull back firmly until fully seated against handpiece.
3. Rotate attachment in direction of arrow on handpiece to align arrow on attachment with lock symbol (Figure 4).
4. Rotate attachment lock sleeve to align arrow with "TUBE" (Figure 1).
5. Insert bearing sleeve into distal end of attachment until fully seated (Figure 5).
6. Rotate attachment lock sleeve to align arrow with "CUTTER" (Figure 1).
7. Gently pull on bearing sleeve to ensure proper engagement.
8. Insert dissection tool into distal end of bearing sleeve and rotate dissection tool slowly until fully seated.
9. Rotate attachment lock sleeve to align arrow with "RUN" (Figure 1).  
**Note:** Additional force is required to seat dissection tool into curved bearing sleeves.
10. Gently pull on dissection tool to ensure proper engagement.

**Note:** Handpiece is now fully functional.

### Bearing Sleeve Adjustment

1. Rotate attachment lock sleeve to align arrow with "ADJUST" (Figure 1).
2. Move bearing sleeve to desired exposure.
3. Rotate attachment lock sleeve to line up arrow with "RUN" (Figure 1).

## Disassembly

1. Rotate attachment lock sleeve to align arrow with "CUTTER" (Figure 1).
2. Remove dissection tool from bearing sleeve.
3. Rotate attachment lock sleeve to align arrow with "TUBE" (Figure 1).
4. Remove bearing sleeve.
5. Hold handpiece firmly in one hand and with other hand push and rotate release sleeve of attachment until arrow is aligned with unlock symbol (Figure 6).
6. Remove attachment from handpiece.

Figure 1

Figure 2

Indicator Bar

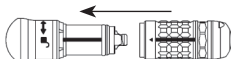
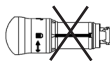
Figure 3

Figure 4

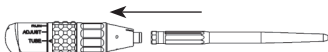
Figure 5

Lock Sleeve

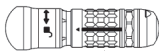
Figure 6

Release Sleeve

## Perforator Driver Attachment

**Note:** When following these instructions hold handpiece and attachments with distal end pointing away from user.

Perforator Driver Attachment is designed to connect to cranial perforators with a Hudson connection.

### Assembly






1. Confirm indicator bar on distal end of handpiece is aligned with unlock symbol (Figure 1) before attempting to install attachment. If not aligned properly, hold handpiece housing firmly with one hand and with other hand rotate distal end in direction of arrow to align indicator bar with unlock symbol.
2. Slide attachment over distal end of handpiece aligning arrow on attachment with indicator bar and unlock symbol (Figure 2). Pull back firmly until fully seated against handpiece.
3. Rotate attachment in direction of arrow on handpiece to align arrow on attachment with lock symbol (Figure 3).
4. To insert cranial perforator, pull back on lock sleeve at distal end of attachment, insert cranial perforator until fully seated and release lock sleeve (Figure 4).
5. Gently pull on cranial perforator to ensure proper engagement.

**Note:** Handpiece is now fully functional.

**Warning:** Follow instructions supplied with cranial perforator before use.

## Disassembly

1. To remove cranial perforator, pull back on lock sleeve at distal end of attachment and remove cranial perforator.
2. Hold handpiece firmly in one hand and with other hand push and rotate release sleeve of attachment until arrow is aligned with unlock symbol (Figure 5).
3. Remove attachment from handpiece.

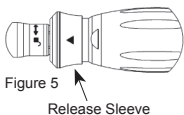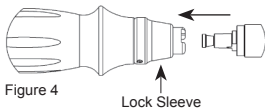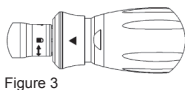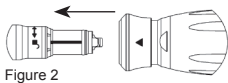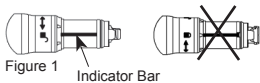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and Anspach are trademarks of DePuy Synthes, Inc.
eG1 is a registered trademark of DePuy Synthes, Inc.
© DePuy Synthes 2021. All rights reserved.
**The Anspach Effort, Inc.**
**Synthes GmbH**