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05.001.095

**Synthes Maintenance Oil, 40 ml,
for Electric Pen Drive and Air Pen Drive**

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**Óleo de manutenção Synthes, 40 ml,
para Electric Pen Drive e Air Pen Drive**

Instruments and implants
approved by the AO Foundation

 **DePuy Synthes**

PART OF THE *JOHNSON & JOHNSON* FAMILY OF COMPANIES

05.001.095

Synthes Maintenance Oil, 40 ml, for Electric Pen Drive and Air Pen Drive

Intended Use

Synthes Maintenance Oil is intended to lubricate surgical instruments, its components and for their maintenance.

General Information

Technically, Synthes Maintenance Oil may support the mechanical properties of surgical instruments and its components, may help to reduce friction and corrosion and therefore heat generation and may help to improve the longevity of the instruments. The oil film is not impairing effective sterilization. Synthes Maintenance Oil is a silicone free product.

Safety Instructions

The Synthes Maintenance Oil is only to be used for lubricating Synthes Power Tool Systems (Electric Pen Drive and Air Pen Drive including its components) and instruments after careful consultation of the instructions for use of the corresponding surgical instrument or power tool. Do not use Synthes Maintenance Oil for Anspach Power Tools.

Synthes Maintenance Oil is just to be applied prior to sterilization.

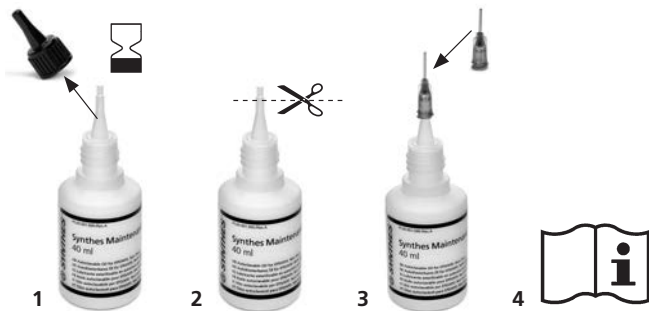
Synthes Maintenance Oil is designed for use by the reprocessing department or the Synthes service center and production personnel.

Do not use Synthes Maintenance Oil after the indicated expiry date.

Instructions for the safe use, disposal and potential hazards associated with the particular chemical mixtures of this product are available for reference in the respective Material and Safety Data Sheet (SE_492001). A copy may be ordered under msds@synthes.com.

Application

1. Check expiry date and remove lid.
2. Remove the safety seal of the dispenser with scissors.
3. For best handling and precise application we recommend the use of a dispense tip (Nordson, green, $d=0.84$ mm).
4. Consult the instructions for use of the corresponding power tool system or surgical instrument for proper use.

Precautions:

- Store in a cool, dry and dark place
- Keep away from direct sunlight
- Read, follow and keep the instructions for use

Explanation of Symbols

Caution

Consult the Instructions for Use before operating the device

The device meets the requirements of directive 93/42/EEC for medical devices. It is authorized by an independent named site for which it bears the CE symbol.

Manufacturer

Product is delivered in an unsterile condition

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All rights reserved. **60099549 SE_484988 AB 10/2016**
© DePuy Synthes Power Tools, a division of Synthes GmbH. 2016.
Synthes GmbH