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05.001.098

**Synthes Maintenance Spray, 400 ml,
for Electric Pen Drive and Air Pen Drive**

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**Spray de manutenção Synthes, 400 ml,
para Electric Pen Drive e Air Pen Drive**

Instruments and implants
approved by the AO Foundation

 **DePuy Synthes**

PART OF THE *JOHNSON & JOHNSON* FAMILY OF COMPANIES

05.001.098

Synthes Maintenance Spray, 400 ml

Intended Use

The Synthes Maintenance Spray is intended to lubricate surgical instruments, its components and for their maintenance.

General Information

Technically, the Synthes Maintenance Spray may support the mechanical properties of surgical instruments and its components, may help to reduce friction and corrosion and therefore heat generation and may help to improve the longevity of the instruments. The oil film is not impairing effective sterilization. Synthes Maintenance Spray is a silicone free product. Do not use Synthes Maintenance Spray for Anspach Power Tools.

Safety Instructions

The Synthes Maintenance Spray is only to be used for lubricating Synthes Power Tool Systems (Electric Pen Drive and Air Pen Drive including its components) and instruments after careful consultation of the instructions for use of the corresponding surgical instrument or power tool.

Synthes Maintenance Spray is just to be applied prior to sterilization.

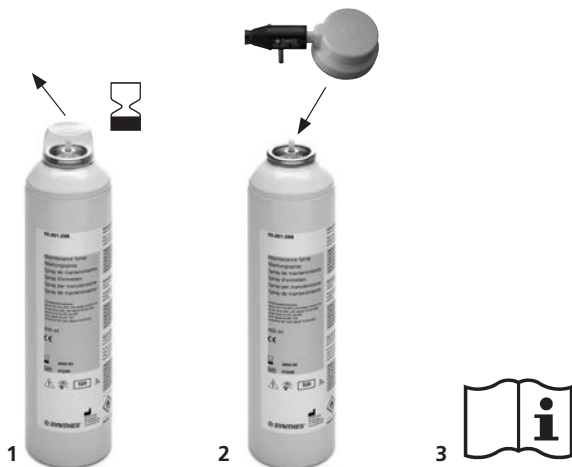
The Synthes Maintenance Spray is designed for use by the reprocessing department or the Synthes service center and production personnel.

Do not use Synthes Maintenance Spray after the indicated expiry date.

Instructions for the safe use, disposal and potential hazards associated with the particular chemical mixtures of this product are available for reference in the respective Material and Safety Data Sheet (SE_492023). A copy may be ordered under msds@synthes.com.

Application

1. Check expiry date and remove lid
2. Attach the specific adapter for the instrument to lubricate:
 - Electric Pen Drive (EPD with adapter 05.001.101)
 - Air Pen Drive (APD, with adapter 05.001.092)
 - Attachments for EPD and APD (with adapter 05.001.102)
 - Screwdriver 90° (with adapter 05.505.002)
3. Consult the instruction for use of the corresponding power tool system or surgical instrument for proper use.

05.001.098

Synthes Maintenance Spray, 400 ml

Precautions:

- Product contains extremely flammable gas under pressure; may explode if heated
- Store in a cool, dry, dark and well-ventilated place
- Keep away from direct sunlight and sources of ignition
- Only use in well-ventilated areas
- Take precautionary measures against static discharges
- Read, follow and keep the instruction for use
- Wet cleaning under mechanical load may dissolve the etiquette of the Synthes Maintenance Spray
- Do not pierce or burn, even after use.

Explanation of Symbols

Caution

Consult the Instructions for Use before operating the device

The device meets the requirements of directive 93/42/EEC for medical devices. It is authorized by an independent named site for which it bears the CE symbol.

Manufacturer

Product is delivered in an unsterile condition

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All rights reserved. **60099550 SE_484989 AB 10/2016**
© DePuy Synthes Power Tools, a division of Synthes GmbH. 2016.
Synthes GmbH