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519.970
Olio speciale Synthes, 40 ml

519.970
Óleo especial Synthes, 40 ml

Instruments and implants
approved by the AO Foundation

 **DePuy Synthes**
PART OF THE *JOHNSON & JOHNSON* FAMILY OF COMPANIES

519.970

Synthes Special Oil, 40 ml

Intended Use

The Synthes Special Oil is intended to lubricate surgical instruments, its components and for their maintenance.

General Information

Technically, the Synthes Special Oil may support the mechanical properties of surgical instruments and its components, may help to reduce friction and corrosion and therefore heat generation and may help to improve the longevity of the instruments. The oil film is not impairing effective sterilization. Synthes Special Oil is a silicone free product.

Safety Instructions

The Synthes Special Oil is only to be used for lubricating Synthes Power Tool Systems and instruments after careful consultation of the instructions for use of the corresponding surgical instrument or power tool. Do not use this product for Electric Pen Drive, Air Pen Drive and Anspach Power Tools.

Synthes Special Oil is to be applied just prior to sterilization.

The Synthes Special Oil is designed for use by the reprocessing department or the Synthes service center and production personnel.

Do not use Synthes Special Oil after the indicated expiry date.

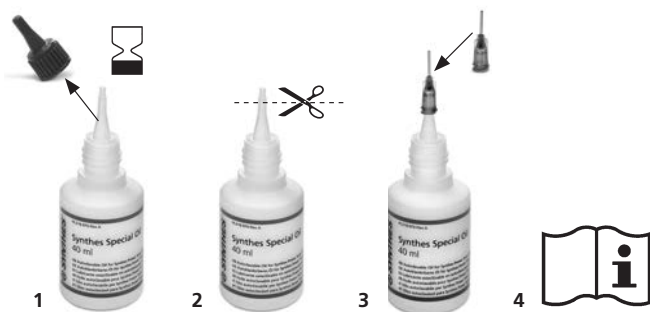
Instructions for the safe use, disposal and potential hazards associated with the particular chemical mixtures of this product are available for reference in the respective Material and Safety Data Sheet (SE_492010). A copy may be ordered under msds@synthes.com.

Application







1. Check expiry date and remove lid.
2. Remove the safety seal of the dispenser with scissors.
3. For best handling and precise application we recommend the use of a dispense tip (Nordson, green, $d=0.84$ mm).
4. Consult the instruction for use of the corresponding power tool system or surgical instrument for proper use.

Precautions:

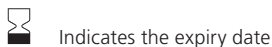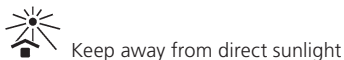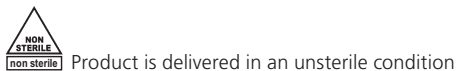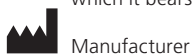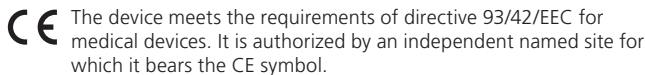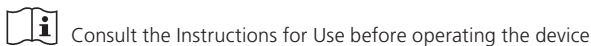- Store in a cool, dry and dark place
- Keep away from direct sunlight
- Read, follow and keep the instruction for use

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All rights reserved. **60099544 SE_484729 AB 10/2016**
© DePuy Synthes Power Tools, a division of Synthes GmbH. 2016.
Synthes GmbH