English

Black Max Standard Attachments

The following instructions are written such that the handpiece and attachments are held with distal end pointed away from user.

Indications for Use
For use with Anspach pneumatic surgical drill systems for cutting and shaping bone.

The following instructions are written such that the handpiece and attachments are held with distal end pointed away from user.

Straight Attachments/Dissection Tool Assembly

- Lift Lever on handpiece. Align notch on attachment with “RELEASE”, which is laser etched on side of handpiece. Turn attachment so notch is aligned with word “SECURE”.
- Insert dissection tool into attachment. Return Lever on handpiece to down position.
- Pull dissection tool to ensure proper engagement.
- Adjust air source until desired air pressure registers on AutoLube gauge.
- Air pressure will not register until foot pedal is depressed.
- Handpiece is operational. Operating pressure is 120 psi (8,2 bar), as indicated on the AutoLube gauge, when using these attachments.

Craniotome, Adjustable Drill Guide & Controlled Depth Attachments/Dissection Tool Assembly

- Lift Lever on handpiece. Insert dissection tool into handpiece. Return Lever to down position. Pull dissection tool to ensure proper engagement.
- Slide attachment over dissection tool. Align notch on attachment with word “RELEASE”. Next turn attachment so notch is aligned with word “SECURE”.
- Handpiece is now operational.
- Operating pressure is 120 psi (8,2 bar), as indicated on the AutoLube gauge, when using these attachmets.

QD Angle Attachments/Dissection Tool Assembly

- Lift Lever on handpiece. Align notch on angle attachment with word “RELEASE”, which is laser etched on the side of the handpiece. Next, turn angle attachment so notch is aligned with word “SECURE” which is laser etched on the side of the handpiece.
- Pull down retaining sleeve on angle attachment and rotate to left when attachment is pointed away from user.
- Insert dissection tool.
- Release retaining sleeve by rotating to left.
- Pull dissection tool to ensure proper engagement.
- Return lever on handpiece to down position.
- Handpiece is now operational.
- Operating pressure is 120 psi (8,2 bar), as indicated on the AutoLube gauge, when using these attachments.

⚠ Warning: Do not operate without irrigation which is mandatory for proper operation.

Warnings

Rx Only: United States Federal law restricts this device to sale by or on order of a physician or other licensed healthcare provider.
• Surgeon is responsible for learning proper techniques in use of equipment; improper use may cause serious injury to user or patient or damage to system.
• Instrument operator and all operating room personnel must wear eye protection.
• Visually inspect for damage before using; do not use if damage is evident.
• Do not use accessories other than those provided by Anspach and specified for use with Anspach systems.
• DO NOT use if the product sterilization barrier or its packaging is compromised.
• DO NOT use, or discontinue use of powered equipment exhibiting excessive temperatures that can cause patient injury (necrosis) and/or user discomfort.
• Use of damaged or improperly maintained power equipment and/or misused powered equipment can result in excessive temperatures.
• To insure equipment operates as designed, read and follow manufacturer’s instructions, including those for proper service and maintenance.
• Do not operate handpiece without an attachment and the corresponding dissection tool. (i.e. Only use Black Max dissection tools with Black Max attachments.)
• Use caution to avoid cutting or tearing gloves while handling dissection tools.
• Dissection tools must be adequately retained within attachment to prevent distal migration, which may cause injury.
• Confirm attachment is proper size for dissection tool and that it is secure.
• Gently pull on dissection tool shaft to ensure it is fully seated and properly installed.
• Do not engage safety mechanism while handpiece is running; doing so makes safety mechanism inoperable.
• Only cut visible areas unless an image intensifier is utilized.
• Delicate structures in proximity to dissection must be thoroughly protected to prevent injury.
• Maintain firm control of instrument at all times.
• Do not bend or use as a lever.
• Use a gentle tapping motion or side-to-side motion and let instrument

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