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05.001.078

Lubrificante para sistemas Anspach®, 110 ml

 **DePuy Synthes**
PART OF THE JOHNSON & JOHNSON FAMILY OF COMPANIES

Instruments and implants
approved by the AO Foundation

 **ANSPACH™**

05.001.078

Lubricant for Anspach® Systems, 110 ml

Intended Use

The lubricant for Anspach systems is intended to lubricate Anspach system attachments during reprocessing, as part of their maintenance.

It is not intended for the lubrication of Anspach BlackMax system or iMRI system attachments.

General Information

Lubrication is MANDATORY to ensure the function of the device as intended by the manufacturer.

The lubricant does not impair effective sterilization. The lubricant for Anspach systems is a silicone-free product and is not made with natural rubber latex.

Safety Instructions

The lubricant for Anspach systems is only to be used for lubricating Anspach power tool system attachments after careful consultation of the instructions for use of the corresponding Anspach power tool system. Use this product only as indicated.

The lubricant for Anspach systems is to be applied just prior to sterilization.

The lubricant for Anspach systems is designed for use by the reprocessing department.

Do not use the lubricant for Anspach systems after the indicated expiry date.

Instructions for the safe use, disposal and potential hazards associated with the particular chemical mixtures of this product are available for reference in the respective Material and Safety Data Sheet (SE_502933). A copy may be ordered under msds@synthes.com.

Application

Do not use lubricant for Anspach systems after the indicated expiry date.

1. Remove cover.
2. Remove protecting cap and install the spray pump provided in the package.
3. Remove the spray pump protective cap and attach the dispenser nozzle provided in the package by pushing it down on the spray pump head.
4. Consult the directions for use of the corresponding Anspach power tool system for proper use.

Precautions:

- Keep away from direct sunlight
- Store in a cool, dry and dark place
- Read, follow and keep the instructions for use
- Wet cleaning under mechanical load may dissolve the etiquette of the lubricant for Anspach systems

Explanation of Symbols

Caution

Consult the Instructions for Use before operating the device

The device meets the requirements of directive 93/42/EEC for medical devices. It is authorized by an independent named site for which it bears the CE symbol.

Manufacturer

Product is delivered in a nonsterile condition

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All rights reserved. **60109948 SE_512018 AC 10/2016**
© DePuy Synthes Power Tools, a division of Synthes GmbH. 2016.
Synthes GmbH